



## Librerie Impianti per Exocad®

### Installazione librerie in Exocad®

Cercare nel vostro Computer in "C" o in Programmi (x86) o in Programmi la cartella denominata Exocad®.

Copiare le cartelle delle Librerie ricevute tramite mail, o fornite dal cd, ed incollarle nella seguente directory nel vostro Pc:

*Exocad/DentalCadApp/Library/implant*

Attenzione : non modificare i nomi delle cartelle delle librerie o i nomi dei file presenti all'interno delle cartelle.

L'attivazione delle librerie avverrà solo dopo il riavvio del programma Exocad®.

Le librerie ricevute sono di 3 tipologie :

- 1) **JDEVOLUTION\_ESTIB\_3.2\_EX**  
Antirotante, per creare Abutment personalizzati, Corone singole anatomiche o ridotte.
- 2) **JDEVOLUTION\_EVTIBNE\_3.7-4.3-5.0-6.0\_ROT**  
Rotazionale, per creare ponti avvitati, strutture toronto avvitate, barre avvitate.
- 3) **JDEVOLUTION\_EVTIB\_3.7-4.3-5.0-6.0\_EX**  
Antirotante, per creare Abutment personalizzati, Corone singole anatomiche o ridotte.

## Implants Library for Exocad®

### Install the Library on Exocad®

Find in your computer ... in " C " or Program Files ( x86 ) or in the Programs folder called Exocad®

Copy the folders of Libraries received by email , or provided by CD , and paste in the following directory in your PC :

*Exocad / DentalCadApp / Library / implant*

Warning : Do not change the folder names libraries or the names of files within folder .  
The activation of the libraries will take place only after restart Program Exocad®

Libraries received are of 3 types :

- 1 ) **JDEVOLUTION\_ESTIB\_3.2\_EX**  
Non-rotating , to create customized abutments ,Single crowns anatomical or reduced anatomical.
- 2 ) **JDEVOLUTION\_EVTIBNE\_3.7-4.3-5.0-6.0\_ROT**  
Rotational , to create bridges screwed , Bridge toronto screwed , screwed bars.
- 3 ) **JDEVOLUTION\_EVTIB\_3.7-4.3-5.0-6.0\_EX**  
Non-rotating , to create customized abutments ,Single crowns anatomical or reduced anatomical.

**JDentalCare s.r.l**  
Via del Tirassegno, 41/N  
41122 Modena Italy

Phone: +39 059 454255  
FAX: +39 059 450045  
Email: [info@jdentalcare.com](mailto:info@jdentalcare.com)